

Ti-System

Reliable Fixation of GTR/GBR membranes



1 Reliable Stability

- Safe, fast and simple fixation of resorbable and non-resorbable membranes¹
- Effective immobilization of membrane and bone graft particulates for undisturbed healing²

2 Titanium Tacks¹

- Hex connection for secure pick up and stable transportation to the surgical site
- Made of Titanium Grade 5
- Available in 3 mm length in pack of 5 units

3 Innovative Tack Design “Nail In – Screw Out”

- Threaded design for rigid fixation and retention
- Easy to remove with the hex screw driver¹

4 Autoclavable Tray

- Space for 15 titanium tacks
- Easy to open thanks to one-hand mechanism
- For machine cleaning and sterilization¹

1. Ti-System IFU latest revision.

2. Fugazzotto PA, Implant and regenerative therapy in dentistry: a guide to decision making. 2009: John Wiley & Sons.


Ti-System: How It Works



1
Sterilization of all components prior to use.



2
Pick the tack from the tray using the application device and place it in-situ using a mallet.

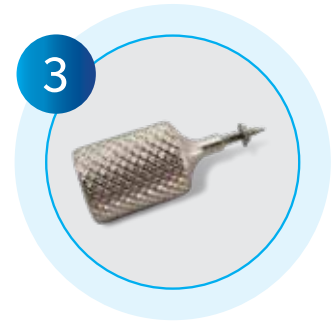

3
At the day of re-entry use the hex screw driver to loosen the tack and remove it with sterile tweezers.

Typically Used For:



Fig. A Lateral Sinus lift procedures to fix CopiOs® Pericardium Membrane.



Fig. B Horizontal ridge augmentations to stabilize bone graft and CopiOs Pericardium Membrane.

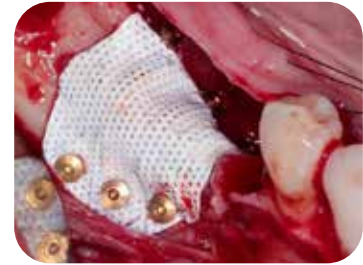


Fig. C Vertical grafting procedures to fix Titanium reinforced OsseoGuard® PTFE Membrane.

Clinical photos. ©2020 Dr. Alejandro Prieto, Spain. All rights reserved. Individual results may vary.

Ordering Information

Item no.	Description
CUR.9000810101	Titanium endpiece, straight
CUR.9000810103	Titanium tack handpiece
CUR.9000810104	Hex screw driver
CUR.9000810105	Sterilization tray (15 tacks max)
CUR.9000810106	Titanium Tack, length 3 mm - pack of 5

Contact us at 1-800-342-5454 or visit zimmerbiometdental.com

Zimmer Biomet Dental
Global Headquarters
4555 Riverside Drive
Palm Beach Gardens, FL 33410
Phone: +1-561-776-6700
Fax: +1-561-776-1272

Unless otherwise indicated, as referenced herein, all trademarks are the property of Zimmer Biomet; and all products are manufactured by one or more of the dental subsidiaries of Zimmer Biomet Holdings, Inc. and marketed and distributed by Zimmer Biomet Dental and its authorized marketing partners. The CopiOs Pericardium Membranes is manufactured by Tutogen Medical GmbH. The OsseoGuard PTFE Membrane is manufactured by Osteogenics Biomedical, Inc. The Ti-System is manufactured by curasan AG. For additional product information, please refer to the individual product labeling or instructions for use. Product clearance and availability may be limited to certain countries/regions. This material is intended for clinicians only and does not comprise medical advice or recommendations. Distribution to any other recipient is prohibited. This material may not be copied or reprinted without the express written consent of Zimmer Biomet Dental. ZB01142 REV A 09/20 ©2020 Zimmer Biomet. All rights reserved.

