

Endobon[®] Xenograft Granules

Regeneration By Design



Endobon Xenograft Granules



Designed To Facilitate Maximum Bone Volume Retention



Endobon Xenograft Granules adhere to one another when hydrated for easy transfer to the defect site.

Small Granules

500–1000 μm particle size preferred for grafting smaller defects, such as extraction sockets.

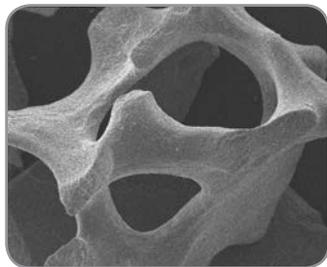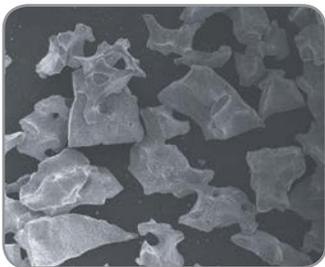

SEM images of Endobon Xenograft Granules at 20x and 100x showing the micro and macro pores in the particles.

Large Granules

1000–2000 μm particle size preferred for grafting large defects, such as sinus elevations.



Conveniently Packaged In Easy-To-Open Dishes.

Value Packs (5 ml and 8 ml) are individually packaged in 1 ml containers for single-unit dosing.



A Bone Graft Substitute

- Bovine-derived hydroxyapatite that has been fully deproteinized by a two-step, high temperature process for protection from bacteria, viruses and prions
- An essentially non-resorbable material that is ideally suited for regeneration of bone defects when effective space maintenance is required
- Osseoconductive due to the interconnecting micro and macro pores for bony integration, which facilitate graft stability and vascular ingrowth¹

Indications

Endobon Xenograft Granules are indicated for the following dental and/or oral surgical procedures:

- Alveolar ridge augmentation/reconstruction
- Filling of bone defects after root resection, cystectomy and apicoectomy
- Filling a socket after tooth extraction
- Sinus elevation

1. Hing KA, Best SM, Bonfield W. Characterization of porous hydroxyapatite. J Mater Sci Mater Med. 1999 Mar;10(3):135-45.

Anterior Ridge Augmentation

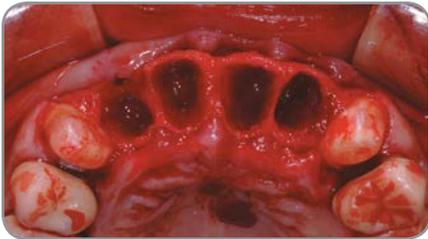


Fig. 1. Extraction sockets of the four maxillary incisors and immediate implant placement.



Fig. 2. Grafting with Endobon Xenograft Small Granules covered by an OsseoGuard® Resorbable Collagen Membrane.



Fig. 3. The soft-tissue flaps were closed and sutured.



Fig. 4. Clinical appearance of soft tissue showing excellent soft-tissue healing after four months.

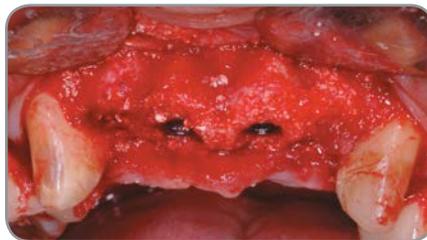


Fig. 5. Clinical appearance of the regenerated site at four months, after removing the remnants of the membrane.



Fig. 6. Occlusal view after four months.

Post-Extraction Defects in the Aesthetic Zone



Fig. 7. Post-extraction defects in the maxilla right central and lateral incisor area.



Fig. 8. Occlusal view of the extraction site defects.



Fig. 9. Facial view of dehiscence defects after implant placement.



Fig. 10. Occlusal view of implants and defects.



Fig. 11. Grafting with Endobon Xenograft Small Granules covered by an OsseoGuard Resorbable Collagen Membrane.



Fig. 12. Regeneration at four months, after removing the remnants of the membrane.

Two Year Histology



Fig. 1. Histologic core of the regenerated bone showing that the Endobon Xenograft Granules integrated with newly formed bone.

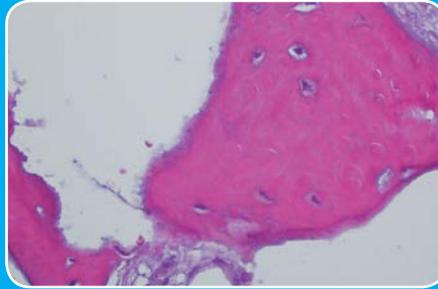


Fig. 2. At 40x magnification, note the newly formed bone (violet) and mature bone (pink) that surround the Endobon Xenograft Granules (white).

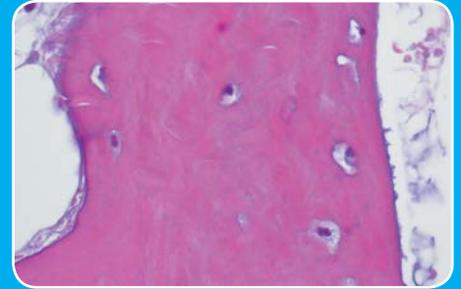


Fig. 3. At 60x magnification, lamellar bone is well formed.

Ordering Information

Endobon Xenograft Granules

Catalog #	Volume	Particle Size
Small Granules		
ROX05	0.5 ml	500–1000 μm
ROX10	1.0 ml	500–1000 μm
ROX20	2.0 ml	500–1000 μm
Large Granules		
ROXLG20	2.0 ml	1000–2000 μm
ROXLG50	5.0 ml*	1000–2000 μm
ROXLG80	8.0 ml*	1000–2000 μm

*Indicates a value pack. Value packs consist of multiple units of 1.0 ml each (5 units or 8 units respectively).



Contact us at 1-800-342-5454 or visit www.zimmerbiometdental.com

Zimmer Biomet Dental
 Global Headquarters
 4555 Riverside Drive
 Palm Beach Gardens, FL 33410
 Tel: +1-561-776-6700
 Fax: +1-561-776-1272

Unless otherwise indicated, as referenced herein, all trademarks are the property of Zimmer Biomet; and all products are manufactured by one or more of the dental subsidiaries of Zimmer Biomet Holdings, Inc., and distributed and marketed by Zimmer Biomet Dental (and, in the case of distribution and marketing, its authorized marketing partners). OsseoGuard Resorbable Collagen Membrane is manufactured by Collagen Matrix, Inc. For additional product information, please refer to the individual product labeling or instructions for use. Product clearance and availability may be limited to certain countries/regions. This material is intended for clinicians only and does not comprise medical advice or recommendations. This material may not be copied or reprinted without the express written consent of Zimmer Biomet Dental. ZB0060 REV A 06/17 ©2017 Zimmer Biomet. All rights reserved.

